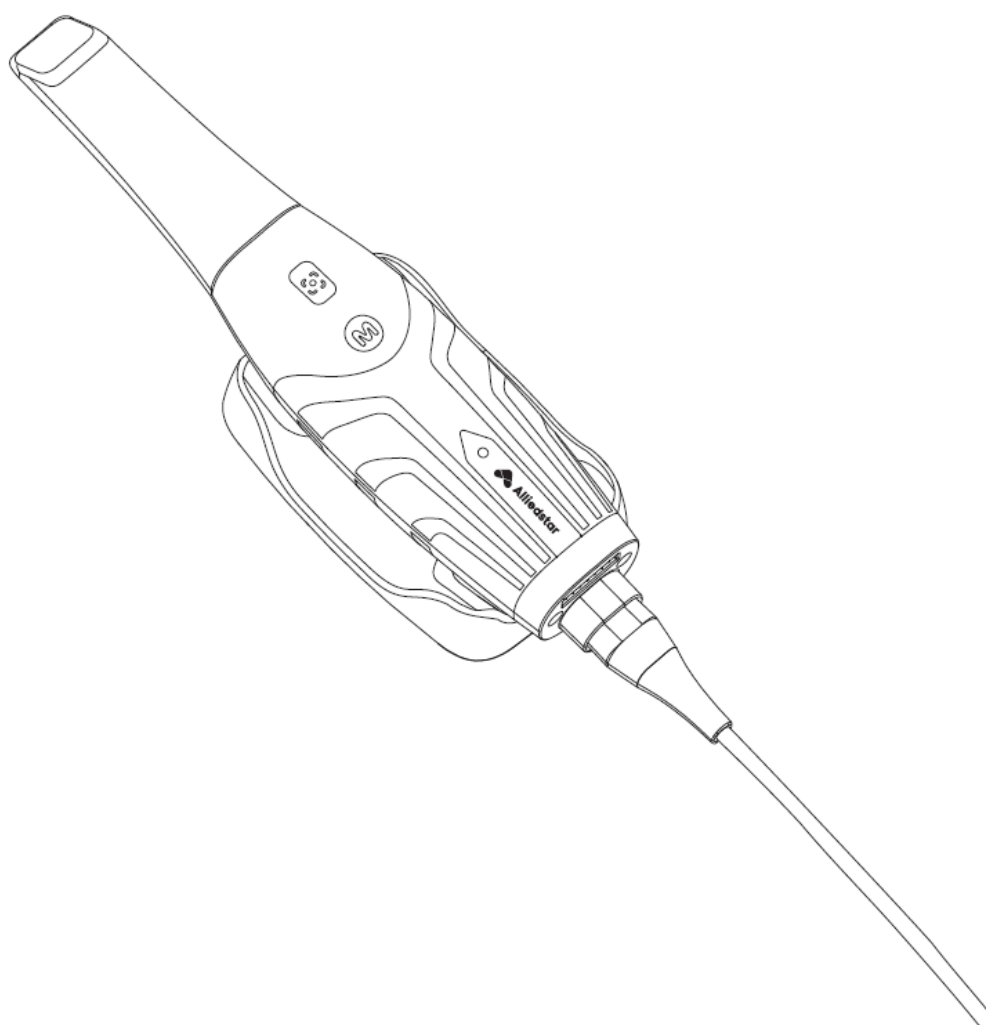
# **AS 260a**

## **Instruction Manual**

**All rights reserved.**

Copyright © 2026 Alliedstar.

The content of this instruction is subject to change.

Alliedstar and its subsidiaries shall not be liable for any indirect damages arising from the preparation, presentation, or use of the information contained in this instruction, including any errors. No part of this instruction can be reproduced without the permission of Alliedstar.

**Trademarks**

All trademarks and registered trademarks are the property of their respective owners.

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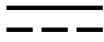
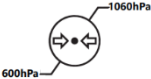
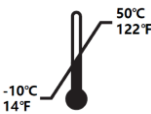
## EN/English

### Special Symbols

The following special symbols are used in this manual to emphasize information or to indicate the presence of potential hazards to personnel and devices.

#### **WARNING**

Avoids injury to yourself or others by following the safety instructions precisely.

#### **CAUTION**

Alerts to a condition that might cause serious damage or cause problems.

#### **NOTE**

Provides extra information and hints.

## Precautions

This manual contains safety instructions, regulatory information, and technical specifications regarding this device.

It is recommended to familiarize yourself thoroughly with this manual to use the device efficiently.

AS 260a should only be used by professionals who have been trained in the system.

In the event of a serious accident while using this product, the user must report it to Alliedstar and the relevant authorities.

### **WARNING**

- The Lens Window at the top of the Handpiece is a precision optical component. When the scanner is not in use, install the Lens Window Protective Cover to protect the Lens Window from damage or dust.
- The mirror inside the Scanner Tip is a delicate optical component. It is important for scanning quality to keep the mirror surface clean and undamaged.
- If a poor image quality is displayed by the software or the video preview is blurry, use a microfiber cleaning swab and unadulterated ethanol to clean the Scanner Tip Mirror and the Lens Window.

## Safety Instructions

The safety instructions include the contents of the warning and safety instructions for scanners, computers, and disposal of discards. Before using this product, be sure to read the safety information and understand its meaning.

### Warnings and Safety Instructions

#### **WARNING ELECTRIC SHOCK**

This device is an electrical device. Do not expose it to water spray as this may result in electric shock or device failure.

#### **CAUTION**

All known residual risks, contraindications, or undesirable side effects are listed in this document. If any serious incident occurs in relation to the device, you must report it to Alliedstar and to the competent authority of your Member State in the European Union.

#### **WARNING**

##### **Scanner**

- Before using the scanner, be sure to read the safety information and understand its meaning.
- The scanner should be used in hospitals and other specialized medical institutions and should not be used in places with high electromagnetic interference intensity, such as near high-frequency surgical device or in RF-shielded rooms of magnetic resonance imaging ME systems.
- The external surface of the device and all its accessories should be inspected first before using the scanner to ensure that there are no rough surfaces, sharp edges or protruding parts on the device to avoid danger.
- The user is responsible for the operation and maintenance of the scanner. The user must attend training on the use of the scanner.
- Maintenance of the scanner should only be performed by professionally trained personnel. Preventive inspections and periodic maintenance other than cleaning, disinfecting, and sanitizing of the scanner are generally not required by the user.

- Do not place objects in the operating area of the device.
- Make sure the scanner is turned off when the device is not in use.
- Do not use the scanner in an oxygen-enriched environment. Do not use this device with flammable anesthetics or combustible materials.
- Do not pull or twist the cables.
- Do not drop the scanner.
- Do not autoclave the scanner.
- Do not expose the scanner to water spray or submerge it in water or disinfectant.
- Do not subject the scanner to severe vibration.
- Do not expose the scanner to direct UV light. The scanner should not be sanitized with UV light.
- Do not stare directly at the LED emitting window.
- Once the Scanner Tip is removed, install the Lens Window Protective Cover to protect the Lens Window.
- Do not remove any scanner component covers. The scanner does not contain user-serviceable parts. Please contact Alliedstar technical personnel for any repairs.
- Do not use cables other than the ones provided with the scanner. Otherwise, it may damage the scanner and adversely affect its safety protection and EMC performance.
- Any other device that does not meet the requirements of IEC 60601-1 should be kept at least 1.5 meters away from the patient.
- If the device malfunctions, it should be turned off, marked as "out of service," and then contact the Alliedstar technical personnel.
- Using components, accessories, cables, and spare parts that are not specified or provided by the device manufacturer may weaken the safety protection of the scanner. It may also cause increased electromagnetic radiation or decreased electromagnetic compatibility, leading to improper operation.
- Do not modify the device without authorization.
- Do not connect additional power sockets or extension cables to the system.
- The maximum temperature of the Scanner Tip may reach 43°C. To avoid excessive temperature, do not prolong the usage time.
- If the device is completely separated from the main power source, please unplug the cable from the USB interface.
- Do not perform maintenance and servicing on the device while it is being used to treat patients.

- Connecting a Programmable Electronic Medical System (PEMS) to an IT network that includes other devices may pose risks to patients, operators, or third parties. Responsible parties should identify such risks and conduct analysis, evaluation, and control measures.
- Patients with oral mucosal diseases, mental illnesses, severe respiratory diseases, asthma, Parkinson's disease, attention deficit hyperactivity disorder (ADHD), and epilepsy are prohibited from using this product.
- Patients with moderate or severe limitation of mouth opening should use this product with caution.

### Computer

- Do not place any device which does not comply with IEC 60601-1 in the immediate vicinity of the patient. Leave at least 1.5 meters distance between the patient and the device.
- The scanner is only intended to be connected to a computer that is at least IEC 60950 / IEC 62368, or equivalent standards certified. Connecting the scanner to other equipment may be hazardous.
- For information regarding data processing systems, computers, and related information, please see the installation guide provided with the computer's display. Leave enough clear space around the computer to ensure that it is properly ventilated.
- Position the screen appropriately to avoid glare caused by indoor or outdoor lighting, in order to achieve optimal image quality and visual comfort.

### Disposal

- This device contains certain materials and compounds that are associated with the manufacturing process of electrical and electronic equipment. Improper disposal of such devices at the end of their lifespan can lead to environmental pollution. Therefore, this device should not be treated as regular household waste but should be taken to designated electrical and electronic waste disposal or recycling centers. For more information on electrical and electronic waste disposal, please contact the relevant regulatory authorities in the local jurisdiction.
- Dispose of the Scanner Tip according to the standard operating procedures or local regulations for the disposal of hazardous medical waste. For additional Scanner Tips, please contact the distributor.

# 1 Product Overview

AS 260a intraoral scanner (the "scanner") is a wired intraoral scanner manufactured by Alliedstar.

The scanner can be used to capture 3D digital impressions of upper jaw, lower jaw, buccal bite registration, and intraoral soft tissues in conjunction with the scanning software, which can be used for data processing, data integration, data export, and data sending, as described in the scanning software instructions.

The scanner is used to acquire digital 3D models in the following modes:

- Upper Jaw
- Lower Jaw
- Buccal bite registration

## 1.1 Intended Use

The digital optical scanning device is used to obtain digital impressions of hard and soft tissues, such as teeth, gums, and mucous membranes, using oral scanning for oral restoration and orthodontic treatment of malocclusion.

The device could be used for both adults and children in clinical practice.

## 1.2 Contraindications

None

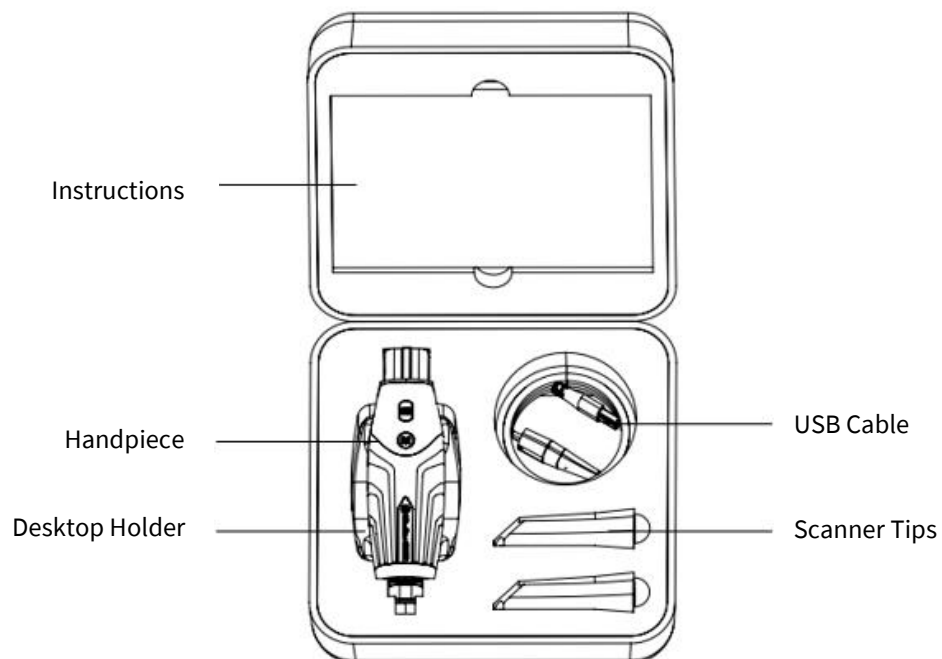
## 1.3 Clinical Benefits

AS 260a enables practitioners to capture digital impressions of the quality and accuracy required for computer-aided design and manufacturing (CAD/CAM) in dental applications that benefit dental practices.

The actual performance of this device is dependent on the degree to which the user implements the training and operational elements. The user is solely responsible for the accuracy, completeness and adequacy of the data acquired.


## 1.4 Unboxing Introduction

The unboxing part is shown in the below picture:


## 1.5 Component Structure

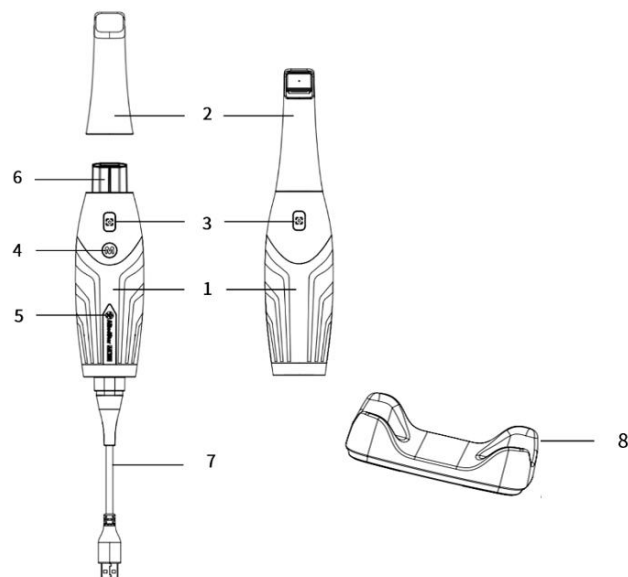The main components of the product are shown in the following parts:

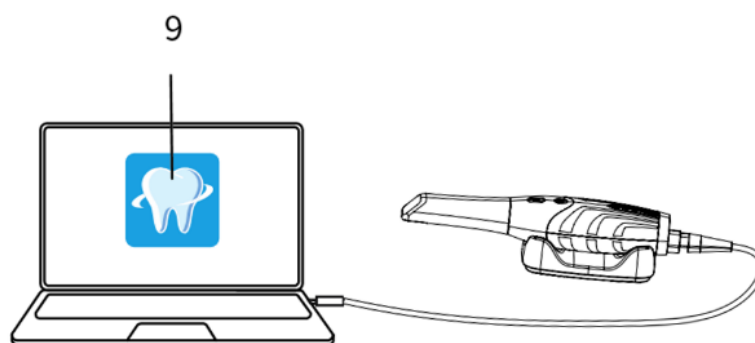The scanner needs to be connected to a computer and the user will need to provide their own computer.

**Component Descriptions**

|                               |                                                                                                                                                                                                                                                     |
|-------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>[1] Handpiece</b>          | Handheld part of the scanner.                                                                                                                                                                                                                       |
| <b>[2] Scanner Tip</b>        | Replaceable application parts in contact with the patient's mouth.                                                                                                                                                                                  |
| <b>[3] Scan Button</b>        | Press the scan button to start scanning.<br>Press the scan button again to end the scanning.                                                                                                                                                        |
| <b>[4] Mode Switch Button</b> | Switch scanning modes.                                                                                                                                                                                                                              |
| <b>[5] Status Indicator</b>   | Slowly flashing: Scanner is booting.<br>Rapidly flashing: Scanner is ready but not selected as the active scanner.<br>Solid on: Scanner is selected as the active scanner.<br>Breathing: Scanner is ready to be connected to the scanning software. |
| <b>[6] Lens Window</b>        | Avoid stains inside the scanner.                                                                                                                                                                                                                    |
| <b>[7] USB Cable</b>          | Provides the power necessary to operate the scanner and facilitates data transfer between the scanner and a computer.                                                                                                                               |
| <b>[8] Desktop Holder</b>     | Support for scanner on desktop.                                                                                                                                                                                                                     |
| <b>[9] Software</b>           | Scanning software for use with scanners.                                                                                                                                                                                                            |

## 1.6 Computer Configuration Requirements

Refer to "8 Technical Specifications" for details on computer system configuration requirements.

### **WARNING**

It must be verified that the computer system configuration meets the computer system requirements of the AS 260a software.

## 2 Installation & Connection

The Installation and Connection section includes a description of the scanner Handpiece, connector (at the end of the USB Cable), and holder installation.

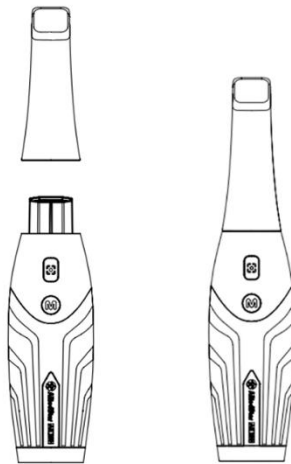
### 2.1 Scanner Installation

The scanner installation is described in the following three parts:

- Handpiece Installation
- Connector Installation
- Holder Installation


#### 2.1.1 Handpiece Installation

1. Remove the Lens Window Protective Cover.
2. Install the Scanner Tip on the Handpiece.

#### 2.1.2 Connector Installation

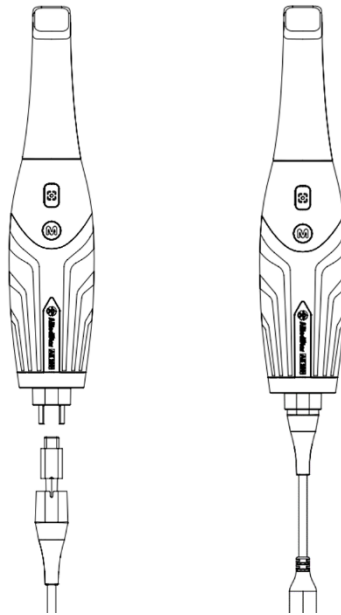1. Remove the Cable Clamp Sleeve Protective Cover at the end of the Handpiece.
2. Insert the USB Type-C Connector.
3. Push the Cable Securing Sleeve toward the scanner to secure the connector.

**CAUTION**

To unplug the USB Type-C Connector, pinch the red-circled area in the following picture with the thumb and forefinger, and then pull off the Cable Securing Sleeve to unplug the USB Type-C Connector.

**2.1.3 Holder Installation**

The holder is used in a desktop type.

### Desktop Holder Installation

To use the holder in a desktop type, please follow the below steps:

1. Selecting the Right Location:
  - Choose a clean, flat surface on a table or desk. This area should be away from any edges to prevent accidental falls and should not be exposed to direct sunlight, high temperatures, or moisture.
  - Wipe the selected area with a dry or slightly damp cloth to remove dust or debris. This helps the Desktop Holder to adhere better to the surface, ensuring stability.
2. Placing the Desktop Holder:
  - Place the Desktop Holder on the prepared surface. Make slight adjustments to the position of the Desktop Holder to ensure it is perfectly flat and stable.

## 2.2 Scanner Connection

Go to <https://support.allied-star.com/download> and download the appropriate installation package and instructions for the product model.

See the ScanPro instructions for software installation details.

Connect the scanner's USB cable to any USB 3.0 port on the computer.

### NOTE

It is important to ensure that the scanner is connected to a USB 3.0 port. If connected to a USB 2.0 port, the scanner may not function properly.

The scanner will automatically power on with the Status Indicator flashing slowly, then enter breathing mode and wait for a connection.

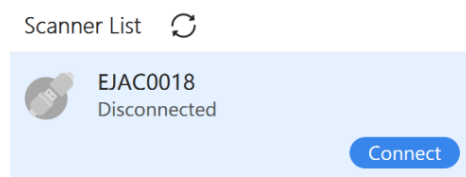
The Status Indicator remains white when the scanner and software are ready.

## 3 Scanning Preparation

### 3.1 Software Interface

To access the software interface, follow the steps below:

1. Launch the software.
2. Register an account and verify the e-mail address.
3. Enter the username and password and click the **[login]** button.
4. Click the green Connect button in the Scanner List to connect the scanner.

5. If the scanner has not been activated, the device activation dialog box will be displayed. Follow the on-screen instructions to complete device activation.
6. Make personalized settings in the **Preferences** options.
7. Press the scan button to scan the 3D model.

### 3.2 Audio Prompts

#### 3.2.1 Scanning Sounds

**Continuous sound:** When scanning sound is enabled, the computer will make a continuous sound when the scanner is scanning images normally.

**Sound interruption:** If the sound is interrupted, it indicates that the data cannot be scanned successfully, and the scanning is stopped.

To continue, return to the previously scanned area until the scanner resumes scanning, and the computer makes a continuous sound.

When the occlusion image is successfully scanned, the computer will also make a short sound.

### 3.2.2 Warning Sounds

A short warning sound is made if the cumulative scanning time for the current case exceeds the recommended threshold, and the computer cannot maintain peak scanning performance or if the scanning process is exposed to bright light.

#### NOTE

To enable the scanning sound, the computer must be equipped with a speaker.

## 3.3 Tooth Preparation

If there is a tooth preparation, shrink the gingiva around the prepared tooth so that the preparation is clearly highlighted.

Dry the teeth thoroughly before starting the scan.

Moderately re-dry the teeth during the scanning process.

## 3.4 Scanner Preparation

**Pre-use Precautions:** Before using the product and accessories, please follow the steps below:

- **Inspect the Scanner Visually**

To visually inspect the scanner for damage or signs of deterioration, please follow the steps below:

1. Inspect the Handpiece Lens Window.
2. Inspect the Scanner Buttons and contacts thoroughly.
3. If damage is found, do not use the scanner and contact Alliedstar technical support personnel.

- **Inspect the Scanner Tip Visually**

To visually inspect the Scanner Tip for damage or signs of deterioration, please follow these steps:

1. Verify that the Scanner Tip is not damaged and that no components are detached.
2. Verify that the Scanner Tip Mirror is not smudged or scratched.
3. Replace the Scanner Tip if signs of deterioration are found.

**WARNING**

- The Lens Window on the Handpiece is a delicate optical component. When the scanner is not in use, install the Lens Window Protective Cover to protect the Lens Window from damage or dust.
- The mirror inside the Scanner Tip is a delicate optical component. It is important for scanning quality that its surfaces are kept clean and undamaged.
- If the image quality displayed by the software is poor or the video preview is blurry, use a microfiber cleaning swab and unadulterated ethanol to clean the Scanner Tip Mirror and Lens Window.

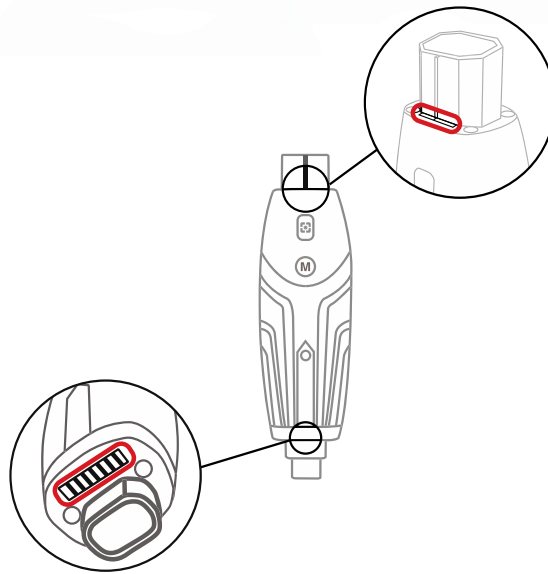
4. Disinfect the scanner and sterilize the Scanner Tip after each use.

**CAUTION**

- The Scanner Tip is not sterilized at the factory.
- Make sure to sterilize the Scanner Tip before using it for the first time.


**CAUTION**

Do not allow any liquid to penetrate the Air Outlet near the Lens Window or the Air Inlet at the rear of the scanner (see the following picture), as this may damage the scanner.

To prepare the scanner, please follow these steps:

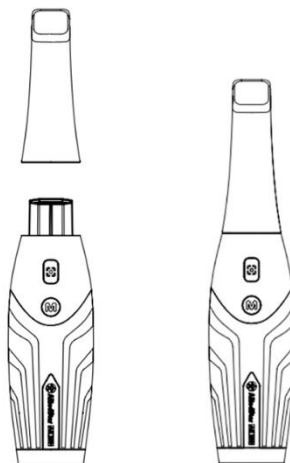1. Make sure the scanner's Lens Window is clean by gently wiping it with a slightly damp, lint-free cloth or lens paper.
2. Assemble the Scanner Tip to the Lens Window, as shown in the following picture.

## 4 Scanning

### 4.1 Scanning Method

#### 4.1.1 Scanning Precautions

To ensure the highest quality and accuracy when using the scanner, please adhere to the following scanning precautions:

- **Initiating the Scan:** Gently position the Scanner Tip directly on the tooth's surface to begin scanning. This will help stabilize the scanner. Activate the scanning sequence by pressing the Scan Button and maintain this action until the 3D image is visibly rendered within the 3D model display area. Once the image appears, methodically and slowly guide the scanner across the tooth's surface and along the dental arch, ensuring comprehensive coverage.
- **Avoidance of Soft Tissues and Contaminants:** It is critical to steer the scanner away from soft tissues and foreign materials, such as saliva. Direct contact with these elements can distort the scan's accuracy and quality, leading to suboptimal outcomes.
- **Sensitivity to Gum Areas:** Exercise caution to prevent the Scanner Tip from contacting sensitive gum areas. Unnecessary contact can cause discomfort to the patient and potentially impact the scan's precision.
- **Bite Scanning Considerations:** It is essential to instruct the patient to bite down naturally, aiming for a centered bite whenever possible to capture an accurate representation of the occlusal surfaces. This position facilitates the capture of a symmetrical view of the occlusal area on both sides of the jaw. Adjust the scanner to ensure it evenly encompasses both the upper and lower teeth, avoiding missed areas that could compromise the scan's completeness.

#### NOTE

During the scanning process, adjust the surgical light so that the light is directed away from the patient's mouth to avoid interference with the imaging.

#### 4.1.2 Recommended Scanning Sequence

Upper/Lower jaw - Lower/Upper jaw – Bites.

### 4.1.3 Recommended Scanning Path

The scanning path is divided into three main scans: occlusal, lingual, and buccal. These scans are designed to comprehensively capture the upper and lower jaw, providing a detailed reconstruction of all surfaces. Follow the steps below for an effective scanning process:

#### 1. Occlusal Scan

Begin the scanning process with the occlusal surface of the first molars. This initial scan lays the foundation for the subsequent scans by capturing the biting surfaces of the teeth.

Carefully move the scanner over the occlusal surfaces of the teeth, ensuring that all areas are adequately covered.

#### 2. Lingual Scan

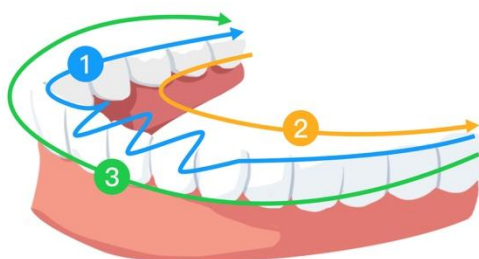
After completing the occlusal scan, proceed to the lingual surfaces. This scan captures the inner surfaces of the teeth. Adjust the scanner to capture these areas comprehensively depending on the jaw being scanned.

#### 3. Buccal Scan

The final scan involves capturing the buccal (cheek-facing) surface. This scan should cover the opposite side of the second scan, ensuring that the outer surfaces of the teeth are thoroughly scanned.

#### 4. Supplementary Scanning

Supplementary scans may be necessary to capture any areas missed in the initial scans or improve the data quality of certain sections.

## 4.2 Operation Method

To scan the 3D model, the steps are generally as follows:

- Scan the upper jaw, lower jaw, and bites
- Refine and check the 3D model
- Complete and save the 3D model

Scanning Procedure:

1. Place the scanner's Scanner Tip on the tooth surface.
2. Wait for the scanner to stabilize and press the Scan Button.
3. Move slowly over the tooth and along the arch until a 3D image appears in the display area of the 3D model.

## 5 Maintenance

The scanner must be handled carefully in accordance with the instructions before use to ensure the highest level of hygiene and safety for each patient. Clean and sterilize the Scanner Tip after each patient.

To minimize the risk of cross-contamination, follow these steps after use for each patient:

- Clean and disinfect the Scanner (See "[5.1 Scanner Cleaning and Disinfection](#)").
- Clean and sterilize the Scanner Tips (See "[5.2 Scanner Tip Cleaning and Sterilization](#)").

The Scanner Tips are available in the following two product models:

| Product Model | Size  | UDI-DI             | Manual Cleaning |
|---------------|-------|--------------------|-----------------|
| TP202         | Large | (01)06973993441034 | Yes             |
| TP203         | Small | (01)06973993441041 | Yes             |
| TP204         | Large | (01)06973993441386 | Yes             |

### CAUTION

This product is only suitable for the above Scanner Tip product models. If you use other Scanner Tips without authorization, situations such as failure to scan will be encountered.

### 5.1 Scanner Cleaning and Disinfection

### WARNING

Read and follow the warnings and personal protection instructions provided in the Safety Data Sheet (SDS) for the sanitizer used to handle the scanner.

Gloves must be worn when cleaning and disinfecting the scanner.

The scanner must be disinfected with a recommended medium level disinfectant solution (with tuberculocidal activity) after each patient.

Do not use a disinfectant containing phenol or iodine-based substances, as this may

damage the surface coating of the scanner.

Do not place the scanner in a pressure steam sterilizer or immerse it in water or a disinfectant solution.

Excessive liquid may damage the scanner.

Do not use cotton balls, pieces of cloth or paper towels soaked in disinfectant to sanitize the scanner.

### 5.1.1 Scanner Cleaning

If the scanner is visibly contaminated with blood or body fluids, it must be cleaned before it is sanitized.

To clean the scanner, please follow these steps:

1. Dampen a lint-free cloth with warm water (do not soak).
2. Use the dampened lint-free cloth to wipe away blood or body fluids.

### 5.1.2 Scanner Disinfection

The scanner must be thoroughly sanitized after use for each patient.

To adequately sanitize the scanner, determine the appropriate length of time to sanitize according to the sanitizer manufacturer's instructions.

#### CAUTION

If there are visible stains on the scanner, it must be thoroughly cleaned prior to disinfection, see "[5.1 Scanner Cleaning and Disinfection](#)".

To sanitize the scanner, please follow these steps:

1. Remove the Scanner Tip.
2. Clean all visible stains (see "[5.1 Scanner Cleaning and Disinfection](#)" ) .
3. Use a commercially available medium level sanitizing cloth. Determine the length of disinfection according to the manufacturer's instructions.

Recommended sanitizing cloths: CaviWipes

**WARNING**

Using a disinfectant that is not recommended may result in damage to the scanner. Wipe all surfaces of the scanner thoroughly. Avoid liquid penetration into the scanner through crevices, Air Inlets and Outlets, etc.

**WARNING**

Do not rinse and allow it to dry naturally.  
After the scanner has dried, wipe any residual disinfectant from the scanner surfaces with a clean, lint-free cloth dampened with water.

## 5.2 Scanner Tip Cleaning and Sterilization

The Scanner Tips supplied by the manufacturer were not cleaned. The Scanner Tip needs to be cleaned before the first use and after each use to clean off surface dirt.

**WARNING**

Wear gloves when handling contaminated Scanner Tips.

Read and follow the warnings and personal protection instructions provided in the Safety Data Sheet (SDS) of any cleaner or sanitizer used to handle the Scanner Tip.

Do not immerse the Scanner Tip in a disinfectant for an extended period of time.

Do not install the Scanner Tip on the scanner until it is thoroughly dry.

Do not use an ultrasonic cleaner to clean the Scanner Tip.

Do not soak the Scanner Tip in an alcohol-based sanitizer.

## Scanner Tip Cleaning

To clean the Scanner Tip manually, please follow the steps below:

1. Rinse off excess dirt from the Scanner Tip with water (2 minutes).
2. Using a soft brush, apply an enzymatic cleaning solvent (e.g., Metrex EmPower) to all surfaces.
3. Rinse with clean tap water (2 minutes).
4. Inspect the Scanner Tip visually and repeat steps 1-3 if it is still not clean.
5. Dry the water on the reflector with lens paper or a lint-free cloth.

### 5.2.1 Scanner Tip Sterilization

The Scanner Tip is not sterilized at the factory and must be sterilized before use.

#### CAUTION

For TP202 and TP203, the Scanner Tip can be autoclaved up to 60 times if the holding time at 134°C is limited to 6 minutes.

For TP204, the Scanner Tip can be autoclaved up to 180 times if the holding time of 134°C is controlled within 6 minutes.

To autoclave the Scanner Tip, please follow these steps:

1. Place the Scanner Tip in a sealed steam sterilization bag.
2. Place the Scanner Tip in a steam autoclave for sterilization.
  - Sterilization temperature should be set to 134°C.
  - Duration should exceed 3 minutes.
  - Duration should be no longer than 6 minutes.

#### CAUTION

For TP203, if you limit the exposure time to 4 minutes at 132 °C, you can autoclave the Scanner Tip up to 60 cycles.

For TP204, if you limit the exposure time to 4 minutes at 132 °C, you can autoclave the Scanner Tip up to 180 cycles.

To autoclave the Scanner Tip, please follow these steps:

1. Put the Scanner Tip into a sealed steam sterilization pouch.
2. Place the Scanner Tip in a steam autoclave for sterilization.
  - Exposure temperature should be set as 132°C.
  - Exposure time: 4 minutes.
  - Drying time: Follow the autoclave manufacturer's recommendation.

## 6 FAQs

| Description                                                                             | Operation                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                 |
|-----------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| There is no connection between the scanner and the computer.                            | <ol style="list-style-type: none"> <li>1. Unplug the USB Type-C connector from the Handpiece and reinsert it.</li> <li>2. Inspect the connection cable for any visible damage. If damage is apparent, replace the connection cable.</li> </ol>                                                                                                                                                                                                                                                            |
| Mismatches and overlaps are visible in the 3D image.                                    | Use the Cut tool to remove mismatched data and excess tissue and rescan.                                                                                                                                                                                                                                                                                                                                                                                                                                  |
| There is a gap or crossover in the occlusal capture.                                    | <ol style="list-style-type: none"> <li>1. Adjust the occlusal pressure.</li> <li>2. Delete the poor occlusion data and rescan.</li> </ol>                                                                                                                                                                                                                                                                                                                                                                 |
| A loss of accuracy or images that do not match well during scanning.                    | <ol style="list-style-type: none"> <li>1. Ensure that the scanner's Lens Window is clean by gently wiping the Lens Window with a lightly dampened lint-free cloth or lens paper.</li> <li>2. Use a slightly damp, lint-free cloth or lens paper to gently wipe any dust or water spots from the Scanner Tip Mirror.</li> <li>3. Make sure the Scanner Tip is securely mounted and there are no black edges or black spots on the live video.</li> </ol>                                                   |
| Difficulty in reconstructing metal preparation teeth.                                   | <ol style="list-style-type: none"> <li>1. Adjust the scanner position (e.g., distance or angle) and scan more areas.</li> <li>2. Adjust the surgical light away from the patient to minimize light scatter.</li> </ol>                                                                                                                                                                                                                                                                                    |
| The inside surface of the Lens Window on the top of the Handpiece appears to be fogged. | <ol style="list-style-type: none"> <li>1. Install a completely dry Scanner Tip on the Handpiece.</li> <li>2. Place the scanner on a stand or a tabletop and wait for the fogging to disappear.</li> <li>3. If the fogging does not completely disappear after the scanner has been sitting for 24 hours, contact the local service provider for assistance.</li> </ol> <p>NOTE: Make sure the Scanner Tip is completely dry before installation, and do not use a rag soaked in disinfectant to clean</p> |

|                                                                                               |                                                                                                                                                                                                                                                                                                                                                                                                          |
|-----------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|                                                                                               | the scanner.                                                                                                                                                                                                                                                                                                                                                                                             |
| After scanning, the scanner and computer are disconnected and do not reconnect automatically. | <ol style="list-style-type: none"><li>1. Check to see if the scanner is turned off.</li><li>2. Press any button on the scanner or remove it from the Holder to turn it on. The power-up process may take several seconds.</li><li>3. Disconnect the USB port from the computer and reconnect it.</li></ol> <p>The scanner should automatically turn on and establish a connection with the computer.</p> |

## 7 Regulatory Information

The AS 260a complies with the following regulations:

- (EU) 2017/745 Medical Device Regulation (MDR).
- FDA Center for Devices & Radiological Health CDRH - Title 21 CFR 872.3661 (USA).
- Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment, 2011/65/EU Annex II and its amendment Directive (EU) 2015/863.

### Compliance with European and International Standards

**EN 60601-1 / IEC 60601-1:** Medical Electrical Equipment, Part 1: General Requirements for Basic Safety and Essential Performance

**ANSI / AAMI ES 60601-1:** Medical Electrical Equipment - Part 1: General requirements for basic safety and essential performance

**CAN / CSA-C22.2 No. 60601-1:** Medical Electrical Equipment - Part 1: General requirements for basic safety and essential performance

**EN 60601-1-2 / IEC 60601-1-2:** Medical Electrical Equipment, Part 1-2: General requirements for basic safety and essential performance – Collateral Standard: Electromagnetic disturbances – Requirements and tests

**EN 80601-2-60 / IEC 80601-2-60:** Medical Electrical Equipment — Part 2-60: Particular requirements for the basic safety and essential performance of dental device.

**EN 62471 / IEC 62471:** Photobiological safety of lamps and lamp systems: Device classification, requirements, and User's Guide

**EN ISO 17664:** Processing of health care products – Information to be provided by the medical device manufacturer for the processing of medical devices

**EN ISO 17665-1:** Sterilization of health care products — Moist heat — Part 1: Requirements for the development, validation, and routine control of a sterilization process for medical devices

**EN 60601-1-6 / IEC 60601-1-6:** Medical Electrical Equipment, Part 1-6: General requirements for basic safety and essential performance - Collateral Standard: Usability

**EN 62366-1/ IEC 62366-1:** Medical devices - Application of usability engineering to medical devices

**EN 62304 / IEC 62304:** Medical device software - Software life cycle Processes

**EN ISO 10993:** Biological evaluation of medical devices

**EN ISO 14971:** Medical devices - Application of risk management to medical devices

**EN ISO 15223-1:** Medical devices - Symbols to be used with medical device labels, labeling and information to be supplied - Part 1: General requirements

**EN ISO 20417:** Medical devices — Information to be supplied by the manufacturer

**ISO 9687:** Dentistry - Graphical symbols for dental device

**AAMI TIR 12:** Designing, testing, and labeling reusable medical devices for reprocessing in health care facilities: A guide for medical device manufacturers

**AAMI TIR 30:** A compendium of processes, materials, test methods, and acceptance criteria for cleaning reusable medical devices

**Classification in accordance with EN / IEC 60601-1**

**Type of protection against electric shock:** N/A

**Degree of protection against electric shock:** Type BF Applied Part

**Mode of operation:** Continuous operation

**Flammable anesthetics:** Not suitable for use in the presence of flammable anesthetics or a mixture of flammable anesthetics with air or oxygen or nitrous oxide.

**Conformity with EN/ IEC 60601-1-2**

**IEC 60601-1-2** EMC requirements and tests, Medical Electrical Equipment including CISPR 11 Group 1, Class B.

### **Electromagnetic Compatibility Precautions**

Medical electrical equipment requires special precautions regarding electromagnetic compatibility (EMC).

Medical devices must be installed and put into service according to the EMC information provided in this documentation.

Other devices can interfere with communications with the AS 260a, even if the device complies with CISPR emissions requirements.

**WARNING**

Portable and mobile RF communications devices (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the AS 260a, including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result.

**Guidance and Manufacturer's Declarations****Guidance and manufacturer's declaration - electromagnetic emissions**

The AS 260a is intended for use in the electromagnetic environment specified below. The customer or user of the AS 260a should ensure that it is used in such an environment.

| <b>Emissions Test</b> | <b>Compliance</b> | <b>Electromagnetic Environment – Guidance</b>                                                                                                                                    |
|-----------------------|-------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| RF Emissions CISPR 11 | Group 1 Class B   | The AS 260a uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. |

**Guidance and manufacturer's declaration – electromagnetic immunity for devices and systems**

The AS 260a is intended for use in the electromagnetic environment specified below. The customer or the user of the AS 260a should ensure that it is used in such an environment.

| <b>Immunity Test</b>                                       | <b>IEC 60601 Test Level</b> | <b>Compliance Level</b>     | <b>Electromagnetic Environment – Guidance</b>                                                                                                  |
|------------------------------------------------------------|-----------------------------|-----------------------------|------------------------------------------------------------------------------------------------------------------------------------------------|
| Electrostatic discharge (ESD)<br>IEC 61000-4-2             | ±8 kV contact<br>±15 kV air | ±8 kV contact<br>±15 kV air | Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%. |
| Power frequency (50/60 Hz) magnetic field<br>IEC 61000-4-8 | 30 A/m                      | 30 A/m                      | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.      |
| Radiated RF IEC 61000-4-3                                  | 3 V/m<br>80MHz – 2.7GHz     | 3 V/m<br>80MHz – 2.7GH      | Environment of a professional healthcare facility.<br><br>WARNING: Portable RF communications                                                  |

|  |  |  |                                                                                                                                                                                                                                                                               |
|--|--|--|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  |  |  | device (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the AS 260a including cables specified by the manufacturer. Otherwise, degradation of the performance of this device could result. |
|--|--|--|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|

For the immunity to proximity fields from RF wireless communications device, the AS 260a is compliant with the test levels specified below, according to IEC 60601-1-2 standard. The customer or user of the AS 260a should be sure that it is used in such an environment.

| Test Frequency (MHz) | Band (MHz)  | Immunity Test Levels                         |
|----------------------|-------------|----------------------------------------------|
| 385                  | 380 – 390   | Pulse modulation 18Hz, 27V/m                 |
| 450                  | 430 – 470   | FM, $\pm 5$ kHz deviation, 1 kHz sine, 28V/m |
| 710                  | 704 – 787   | Pulse modulation 217Hz, 9V/m                 |
| 745                  |             |                                              |
| 780                  |             |                                              |
| 810                  | 800 – 960   | Pulse modulation 18Hz, 28V/m                 |
| 870                  |             |                                              |
| 930                  |             |                                              |
| 1720                 | 1700 – 1990 | Pulse modulation 217Hz, 28V/m                |
| 1845                 |             |                                              |
| 1970                 |             |                                              |
| 2450                 | 2400 – 2570 | Pulse modulation 217Hz, 28V/m                |
| 5240                 | 5100 – 5800 | Pulse modulation 217Hz, 9V/m                 |
| 5500                 |             |                                              |
| 5785                 |             |                                              |

### Accessories

Using cables or accessories other than those specified, except those sold by the device manufacturer, as replacement parts for internal components may result in increased emissions or decreased immunity of the medical device.

**Other Devices****WARNING**

Use of this device adjacent to or stacked with other devices should be avoided because it could result in improper operation. If such use is necessary, this device and the other device should be observed to verify normal operation.

## 8 Technical Specifications

### Product Model

AS 260a

### AS 260a Technical Specifications

| Components                                  | Technical Specifications                                                                                                                           |                                                                                                                    |                                                                                                                      |
|---------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------|
| <b>Weight</b>                               | Handpiece: 175g<br>Tip Large - Sterilization (TP202): 14g<br>Tip Small - Sterilization (TP203): 12g<br>Tip Large - Sterilization (TP204): 14g      |                                                                                                                    |                                                                                                                      |
| <b>Color</b>                                | 3D full color                                                                                                                                      |                                                                                                                    |                                                                                                                      |
| <b>Connectivity</b>                         | USB 3.0                                                                                                                                            |                                                                                                                    |                                                                                                                      |
| <b>Power source</b>                         | USB 3.0 5V, 900mA                                                                                                                                  |                                                                                                                    |                                                                                                                      |
| <b>Field of view</b>                        | Tip Large - Sterilization (TP202): 16mm x 14mm<br>Tip Small - Sterilization (TP203): 12mm x 12mm<br>Tip Large - Sterilization (TP204): 16mm x 14mm |                                                                                                                    |                                                                                                                      |
| <b>Minimum computer system requirements</b> |                                                                                                                                                    | <b>Desktop</b>                                                                                                     | <b>Laptop</b>                                                                                                        |
|                                             | <b>CPU</b>                                                                                                                                         | <ul style="list-style-type: none"> <li>10th Generation Intel® Core™ i5-10600</li> <li>AMD Ryzen™ 5 3600</li> </ul> | <ul style="list-style-type: none"> <li>11th Generation Intel® Core™ i5-11400H</li> <li>AMD Ryzen™ 7 5700U</li> </ul> |
|                                             | <b>Graphics</b>                                                                                                                                    | NVIDIA GeForce GTX 1660 Ti 6 GB                                                                                    | NVIDIA GeForce GTX 2060 (Notebook) 6 GB                                                                              |
|                                             | <b>RAM</b>                                                                                                                                         | 16 GB                                                                                                              |                                                                                                                      |
|                                             | <b>Disk</b>                                                                                                                                        | 512 GB SSD                                                                                                         |                                                                                                                      |
|                                             | <b>Display</b>                                                                                                                                     | 15.6" FHD (1920x1080)                                                                                              |                                                                                                                      |
|                                             | <b>OS</b>                                                                                                                                          | Win 10 (build 18362+) / Win 11, 64 bit                                                                             |                                                                                                                      |
|                                             | <b>Other</b>                                                                                                                                       | USB 3.0 port                                                                                                       |                                                                                                                      |
|                                             | <b>Optional</b>                                                                                                                                    | Touch screen                                                                                                       |                                                                                                                      |

**CAUTION**

It is MANDATORY to verify that your system configuration is compatible with the computer system requirements of the AS 260a.

**Environmental Requirements**

| Components                                      | Environmental Requirements |
|-------------------------------------------------|----------------------------|
| Operating temperature                           | 15°C ~ 30°C                |
| Transport and storage temperature               | -10°C ~ 50°C               |
| Operating relative humidity                     | 10% RH ~ 65% RH            |
| Transportation and storage relative humidity    | 10% RH ~ 95% RH            |
| Operating atmospheric pressure                  | 70 KPa ~ 106 KPa           |
| Transportation and storage atmospheric pressure | 60 KPa ~ 106 KPa           |

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**Alliedstar Medical Equipment Co., Ltd.**
**Etkon GmbH**